**GEBRAUCHSANLEITUNG
DIRECTIONS FOR USE
NOTICE D'UTILISATION
GEBRUIKSHANDLEIDING
MANUALE PER L'USO
MANUAL DO UTILIZADOR
MANUAL DE INSTRUCCIONES**

BREEZY 300

**SUNRISE
MEDICAL**

Fig. 1

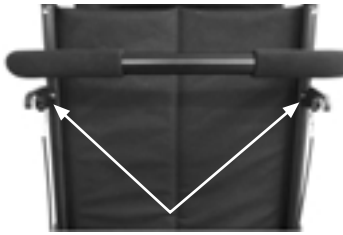
Fig. 2

Fig. 3

Fig. 4

Fig. 5

Fig. 6

Fig. 7

Fig. 8

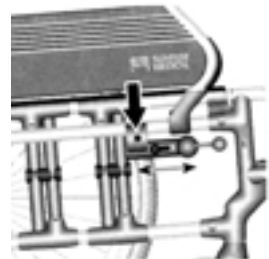
Fig. 9

EINLEITUNG

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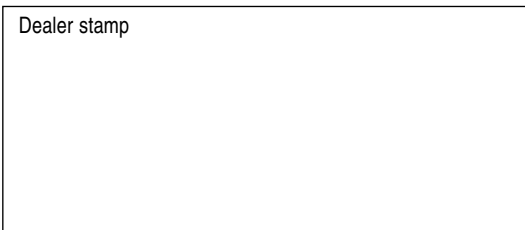
INTRODUCTION

Thank you for choosing a SUNRISE MEDICAL product. In view of the broad range of components and accessories available for all SUNRISE MEDICAL wheelchairs it is in your interest to consult a health professional or authorised SUNRISE MEDICAL distributor. These wheelchairs are designed for persons with a physical handicap, to be used only for transport. These wheelchairs are suitable for use both indoors and outdoors (Class B product). The manufacturer reserves the right to modify, without prior notice, weights, dimensions, or any other technical features described in this manual. All figures, measures, and capacities included in this manual are approximate and are not specifications.

Models: 305, 312, 315, 310, 321, 341

Sunrise Medical LTD.
 Sunrise Business Park
 High Street, Wollaston
 West Midlands DY8 4PS
 England
 Phone: 1384 44 66 88
 Fax: 1384 44 66 99

Sunrise Medical is ISO 9001 certified, which ensures quality at all stages of the development, and production of this wheelchair.


This product is manufactured to comply with the radio interference requirements of EEC directive 89/336/EEC Medical Device Directive 93/42/EEC.

INDEX

A.CHARACTERISTICS	6	E.ADJUSTMENTS	8
B.SAFETY	7	1. Footrest	
C. HANDLING	7	2. Wheel locks	
D. ASSEMBLY	8	F.CONTROLS AND MAINTENANCE ...	8
1. Footrest		G.SPECIFICATIONS	9
2. Armrest		H. MEASURES	9
3. Head-rest		I. GUARANTEE	9
4. Stabilising bar			

A. CHARACTERISTICS

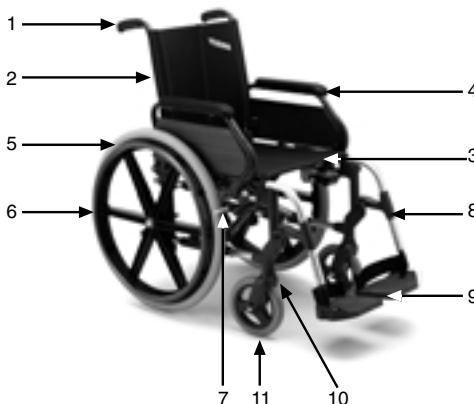1. Handgrip
2. Back upholstery
3. Seat upholstery
4. Armrest upholstery
5. Push rim
6. Propulsion wheel
7. Wheel lock
8. Footrest
9. Footplate
10. Fork
11. Castor

B. SAFETY TIPS

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed.

- Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents.
- As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. You should always wear clothing which is light-coloured when it is dark, so that you can be seen more easily. Also, make sure that the reflectors are not covered.
- Be careful during your first ride in this wheelchair. Get to know your wheelchair. Test what effect changing the centre of gravity would have on your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.
- CAUTION: The wheellocks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheellocks to prevent such rolling. Always use both wheellocks; otherwise, your wheelchair could tip over. The wheel locks have not been designed to be used as brakes for a moving wheelchair.
- Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants. They should grip the wheelchair at securely mounted parts. A wheelchair should never be lifted with an occupant; it should only be pushed.
- Before each use, the following should be checked: Quick-release axles (if fitted) on the rear wheels / Tyres, tyre pressure, and wheellocks. Wheellocks will not function properly if tyre pressure is too low or the interval between tyres and wheellocks is too large.
- The maximum user weight is 100 kilograms.
- Transportation in vehicles: This wheelchair has been crash tested in its standard configuration. However, due to safety risks Sunrise Medical do not recommend the transportation of people in their wheelchairs. Should a person need to be transported in wheelchair on transport systems, the following should be adhered to:
 1. The wheelchair must be positioned forward facing.
 2. The Unwin restraint system, model 4 point WWR/ATF/K/R or equivalent specification must be used.
 3. The restraint system must be fixed as per labelling on the wheelchair. For tie down points on the wheelchair, please look for the tie-down symbols on the wheelchair.
 4. A head restraint must be fitted and suitably positioned at all times during transportation. (Unwins HR models recommended).
 5. If possible remove seat cushion to create a lower centre of gravity.
 6. The chair is crash tested and meets the requirements of ISO 7176-19

C. HANDLING

1. Opening: Rest both hands on each of the seat upholstery support rails and press down until the seat and back upholsteries become taut. Lower the footplates into the using position. As a safety measure, the footplates will always be in an upright position before sitting or getting out of the chair in a forward movement. Warning: Do not place your fingers between the seat upholstery and the seat tubes when opening or folding the wheelchair

2. Folding: First flip up the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the out-side and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing.

3. Folding back-rests (models 315 and 310) (fig. 1) To fold down the back-rest, press the black buttons on the components on the back-rest tubing and pull back the push handles.

4. Reclining back (4° - 55°) (Fig. 2): By pulling the 2 levers at the same time the backrest is released and can be set to the required position. Releasing the 2 levers will lock the backrest automatically in the new position. To bring the backrest back to the upright position, just move it (no release necessary) to the required position.

Caution: The reclining back is only to be used in combination with the active anti-tips setting

Caution: The reclining back is only to be used in combination with a stabilizer bar. Please always make sure that the stabilizer bar is locked in place on both sides.

Caution: Before folding the wheelchair, make sure that the stabilizer bar is removed

Caution: In order to avoid any reduction in the cable function, you must make sure that the cables are not completely taut.

Caution: When adjusting the back angle, be careful not to get your fingers caught.

Caution: Before adjusting the back angle, put the anti tip tubes in to the active position

5. elevating legrest (Fig. 3) To elevate: Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place. To lower: Lift your lower leg and press the release button to unlock the legrest to lower it. As soon as you release the button, the legrest will click back into place. CAUTION: The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the casters. CAUTION: Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

D. ASSEMBLY

1. Footrests (Fig. 4): The footrests can be rotated inwards or outwards by pressing the release latch and turning the footrest in the desired position. To remove the footrest, press the release latch, rotate the footrest approximately 90° and lift.

2. Armrests (Fig. 5): The armrest can either be flipped back or can be detached from the chair. To flip arms back, press the release latch and pull arm toward back of the wheelchair. To detach the armrest press the button located on the back of the armrest and lift it. CAUTION: Make sure that the armrest is properly locked into place. Never lift the wheelchair by the armrests as these parts are detachable and lifting the wheelchair by them may cause damage to the chair and/or injury to the user.

3. Head-rest (Fig. 6): To install the head-rest, push the head-rest tubes into the tubes at the top of the back-rest until the locking pins click into position in the holes. To remove it, press the locking pins and pull the head-rest up until it comes off.

4. Reclining back-rest stabilising bar (Fig. 7): Installation: Insert the bar into the tubes on the back of the back-rest. Press the locking pins and hold them down while inserting the stabilising bar, until the pins lock into place in the holes in the bar. To remove the bar, reverse the process.

E. ADJUSTMENTS

1. Footrest heights (Fig. 8) To adjust the height of the footplates, loosen the retaining bolt in the frame tube, reposition footplate and tighten bolt. For these adjustment use a 10 mm. wrench. A minimum interval of 2.5 centimetres from the ground should always be maintained.

2. Wheel locks (Fig. 9) To adjust, loosen the screw on the top of each clamp. Using a 5 mm. allen key, turn one of the screws counter clockwise one-quarter turn. Repeat the same process with the second of the two screws. Alternately loosen the screws until both screws are removed. Slide clamp toward the rear wheel until the knurl embeds into the tyre approximately 5 mm. when in the locked position. Tighten screws.

F. CONTROLS AND MAINTENANCE

MAINTENANCE: In addition to the maintenance and checks, general cleaning of both frame and upholstery can be undertaken. To maintain its appearance the upholstery should be cleaned regularly. Light soiling can be removed by sponging with a mild detergent and warm water, wiping in the direction of the texture. Rinse off with clean water before allowing to dry. Do not put upholstery in a dryer. If dirt is ingrained, the surface should be scrubbed with a soft brush. The frame should be cleaned regularly with a non-abrasive cleaner followed by polishing with a non-abrasive automotive wax.

CONTROLS: A periodic check of the following procedures will help sustaining the life and efficiency of your wheelchair.

Weekly: Check de pressure of the wheels following the instructions indicated on them. Check that the cross brace's central screw is tight and in good condition / Clean the upholstery with soapy water or adequate products / Clean the metallic parts of the chair with a dry cloth or with special products for chromium plated or painted surfaces / Check the wheel locks and adjust if necessary / Check wear of front and rear tyres.

Quarterly: Check and adjust if necessary: Castor play / Play in the castor forks / Push wheels' play / Offcentering of the push wheels / Lubricate with industrial oil or grease all parts subject to friction or wear / Retighten screws and nuts / A general revision of the chair's condition, repairing whatever is necessary.

Yearly : We recommend a complete revision once a year in a specialized establishment

G. SPECIFICATIONS

Footrests: 70° Swing away, detachable and height adjusting. Elevating leg-rests optional

Armrests: Detachable and flip back

Standard Castors: 200 mm solid

Standard Rear Wheels: 600mm pneumatic in models: 305, 315 and 321. 300 mm. pneumatic in models 312, 310 and 341

Wheel Locks: high push-to-lock.

Spare Parts: Each wheelchair component is available as an spare part.

Accessories: **Foot-rests:** elevating. **Wheels:** Solid tyres. **Wheel locks:** extension handles and drum brake.

Others: belt, anti-tip tubes, table, crutch and dripping device supports.

H. STANDARD MEASURES

Model	Seat width	Seat height	Seat depth	Backrest height	Overall width	Overall length	Weight Without footrests
305	36-48	51	42	49	56-68	106	11,8 kg.
312	36-48	51	42	49	50-62	94	11,3 kg.
315	36-48	51	42	49	56-68	106	11,8 kg.
310	36-48	51	42	49	50-62	94	11,3 kg.
321	36-48	51	42	79	56-68	114	13,8 kg.
341	36-48	51	42	79	52-64	103	13,3 kg.

I. GUARANTEE

SUNRISE MEDICAL guarantees these wheelchair against a defect in materials or manufacture for the period established by the current guarantee law in each country. The following are not guaranteed: tyres, rubber parts, upholstery, armrest pads. SUNRISE MEDICAL will repair or replace according to request providing such repair or replacement is authorized by an established SUNRISE MEDICAL agent and that the part or parts in question are returned within the guarantee period. The cost of labour in fitting or repairing such parts, and of the shipping involved are not included in the guarantee. SUNRISE MEDICAL will not guarantee parts that are damaged owing to inappropriate fitting and/or repair by the owner of the wheelchair, negligence, accident, or improper use. The parts modified or the service rendered by persons not authorized by SUNRISE MEDICAL are not guaranteed. SUNRISE MEDICAL will not guarantee the frame or parts of the wheelchair damaged when the maximum weight that the wheelchair will support. i.e. 100 kg., has been exceeded.

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Sunrise Medical Srl.